## Finding The Right Fit

Proper mask fit is critical to ensuring that the ComfortGel will meet the patient's needs.


Below are some guidelines to follow when fitting a patient with a ComfortGel mask:

## Landmarks


- a.) Sides of nostrils
- b.) Bridge of nose zone
- c.) Below the nose tip, above the lip


## Directions

- 1. Begin with sizing gauge. The integrated sizing gauge on the package:
  - Allows for quick and easy fitting.
  - Cuts down on mask waste by letting you determine the right mask size before you open the bag.
- 2. Check landmarks.
- 3. Choose the smallest mask that fits without pinching or obstructing the nostrils.

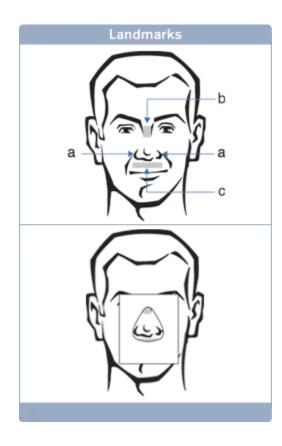

**Note:** For proper mask sizing print out this page at 100%. Do NOT select the 'Shrink oversized pages to paper size' option. The box below should measure 1 inch square.

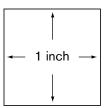

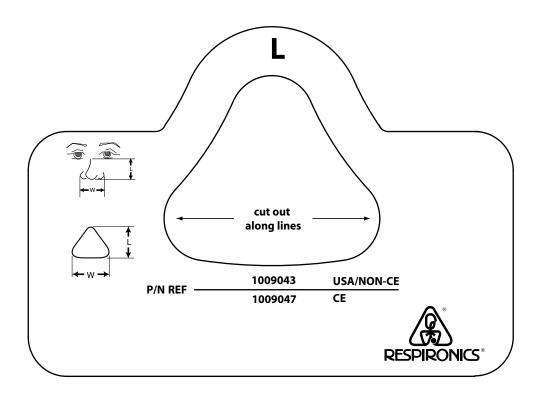





**Note:** For proper mask sizing print out this page at 100%. Do NOT select the 'Shrink oversized pages to paper size' option. The box below should measure 1 inch square.

